

## EasyCare Therapy Click System

The Therapy Click System is a therapeutic aid and a starting point to change breakover and heel elevation. Both the 5 degree and 10 degree bases have been designed with extra depth to give professionals the ability to customize to each unique case. Breakover position and heel height are quickly changed with a rasp or grinder. Add additional height with adhesive products.

### Installation Instructions

1. Get the horse comfortable in an EasyCare hoof boot with an EVA pad or the packing of your choice.
2. Apply the Therapy Click System base on the EasyCare hoof boot of your choice.
3. If needed, trim the Click base width to match boot.
4. Customize the breakover and heel height of the EasyCare Therapy Click to match each unique case (grinder, rasp or addition of adhesive).
5. Keep the horse comfortable. The Click bases can be added or removed from the boot without removing the boot from the horse.



### Compatible with the following EasyCare hoof care products:

- Easyboot Rx
- Easyboot Cloud
- Easyboot Stratus
- Easyboot Flip Flop
- Original Easyboot
- Easyboot Epic
- Easyboot Back Country
- Easyboot Glove
- Easyboot Glue-On

*Note: Therapy Click System is not designed to work with wide sizes.*



Insert 4 Self-Tapping Screws



Therapy Click 5 Degree



Therapy Click 10 Degree

### EasyCare Therapy Click System Size Chart

Easyboot Glove, Glue-On & Back Country	00	00.5	0	0.5	1	1.5	2	2.5	3	3.5	4	4.5	5
Click Size	B	B	B	C	C	C	C	D	D	D	D	E	E
Easyboot Flip Flop	0	0.5	1	1.5	2	2.5	3						
Click Size	B	C	C	C	C	D	D						
Original Easyboot & Easyboot Epic	Pony	000	00	0	1	2	3	4	5	6	7		
Click Size	NA	NA	B	C	C	D	D	E	NA	NA	NA		
Easyboot Rx	00	0	1	2	3	4	5	6	7				
Click Size	C	C	C	D	D	E	NA	NA	NA				
Easyboot Cloud & Stratus	00	0	1	2	3	4	5	6	7	8			
Click Size	B	C	C	D	D	E	E	NA	NA	NA			